
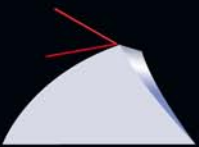
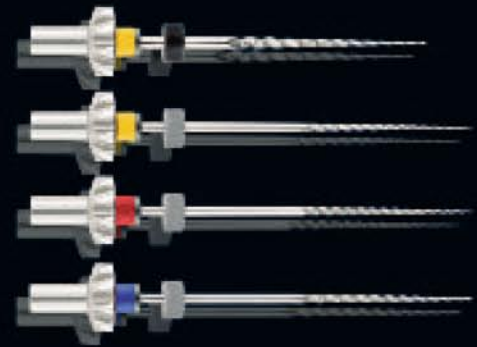


Endodontic Treatments

Simple, Safe, Efficient

HERO Shaper®

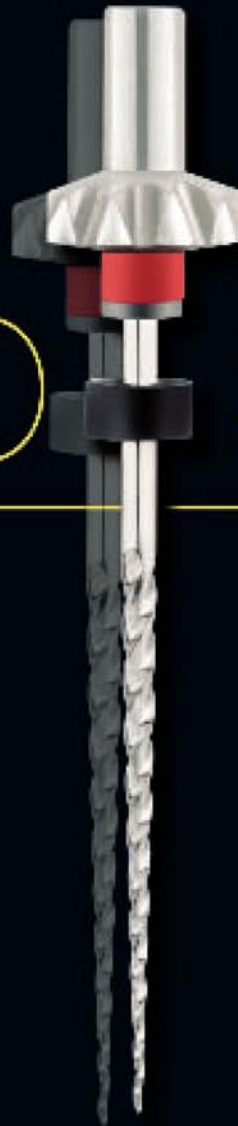

Positive rake angle
for excellent debris
evacuation.



Big inner core
for excellent resistance
to breakage.



3 edges
for better force
balance.



Adapted pitch
Flexibility / Strength /
No screw-in effect.



Anti Breakage Control
the instrument unwinds to
prevent fracture.

Yellow Sequence Complex cases

1 n° 20 -.06 taper
Bring the HERO Shaper® n° 20 -.06 taper to the 2/3 of WL.

2 n° 20 -.04 taper
Bring the HERO Shaper® n° 20 -.04 taper to WL.

3 n° 25 -.04 taper
Bring the HERO Shaper® n° 25 -.04 taper to WL.

4 n° 30 -.04 taper
Bring the HERO Shaper® n° 30 -.04 taper to WL.

Red Sequence Intermediate cases

1 n° 25 -.06 taper
Bring the HERO Shaper® n° 25 -.06 taper to the 2/3 of WL.

2 n° 25 -.04 taper
Bring the HERO Shaper® n° 25 -.04 taper to WL.

3 n° 30 -.04 taper
Bring the HERO Shaper® n° 30 -.04 taper to WL.

Blue Sequence Simple cases

1 n° 30 -.06 taper
Bring the HERO Shaper® n° 30 -.06 taper to the 2/3 of WL.


2 n° 30 -.04 taper
Bring the HERO Shaper® n° 30 -.04 taper to WL.

2 concepts: the choice

InGeT® Version
(Integrated Gear Technology)
The driving gear is part of the rotary file.



- || Miniaturized Head.
- || Ergonomics.
- || Visibility.
- || Accessibility.

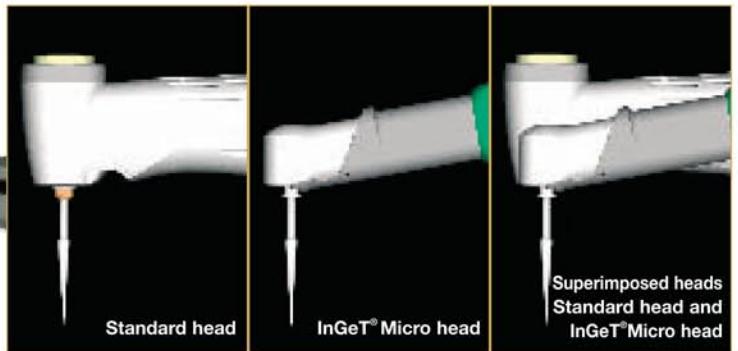

Classics Version
(Ø 2,35 shaft) to be fitted on an endo reducing contra-angle (AXS Endo® for instance).



AXS Endo® contra-angle

Your distributor