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MM-MTA – MM-MTA Caps Endodontic Repair Cement

MM-MTA is an endodontic repair cement. MM-MTA powder consists of very fine hydrophilic particles of several mineral oxides. After contact with MM-MTA liquid it forms a gel that hardens to an impermeable barrier.

MM-MTA is delivered in MM-MTA Caps. MM-MTA Caps are easily activated and the content of the capsule is easily ejected out with the MM-MTA Gun. Capsule mixing (mixing time 30 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/min.

1. Intended uses

- Repair of root perforations during root canal therapy.
- Root-end fillings.
- Pulp capping.

2. Contraindications

Not known.

3. Side effects

Not known.

4. Activation and Mixing (see Instruction for MM-MTA Caps)

Activate and mix the MM-Caps according to the information in the MM-MTA Caps instruction.

Mixing time for the MM-Caps is 30 seconds.

Attention:

Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting which may impair or prevent application of the material. The material must be extruded within 10 seconds after the end of mix.

To prevent dehydration during setting, apply MM-MTA intraorally immediately after mixing.

Working time of MM-MTA is approx. 2 minutes (at 23°C).

Setting time of MM-MTA is about 20 minutes. Net setting time is about 2 hours.

5. Application

5.1. Repair of root perforations

Place rubber dam and clean the root canal system using intra-canal instruments and irrigate with NaOCl. Dry the root canal with paper points and isolate the perforation. Fill the canal space from the apical part up to the perforation.

Mix MM-MTA as described under point 4 and extrude it on a clean glass plate.

Apply MM-MTA with suitable instruments into the perforation site and condense it.

Check the position of MM-MTA in the root canal by an X-ray. If an adequate barrier has not been created, rinse MM-MTA out of the canal and repeat the procedure.

After 20 minutes, obturate the remaining canal space. MM-MTA repair material remains as a permanent part of the root canal filling.

If the treatment is made in 2 sessions, take a damp cotton pellet, remove excess moisture from the pellet and place it in the canal. Seal the access preparation with a temporary restoration.

5.2. Root-End Fillings

Create an access to the root-end and resect the root with a surgical bur.

Use an ultrasonic tip to prepare a class I root-end cavity preparation to a depth of 3-5 mm.

Isolate the area and dry the root end cavity with paper points.

Mix MM-MTA as described under point 4 and extrude it on a clean glass plate.

Apply MM-MTA with suitable instruments and condense it using a small plugger.

Remove excess cement and clean the surface of the root with a moist piece of gauze.

Confirm placement of the MM-MTA repair material with an X-ray. The MM-MTA repair material remains as a permanent part of the root canal filling.

5.3. Pulp Capping

Place rubber dam and prepare the access cavity. If caries is present, remove it. Rinse cavity and exposed pulpal areas with 3-5% NaOCl.

Mix MM-MTA as described under point 4 and extrude it on a clean glass plate.

With a suitable instrument apply a small amount of MM-MTA over the exposed pulp and remove excess moisture with a dry cotton pellet.

Place a small amount of a light cure liner and light cure.

Etch the remaining cavity walls according to the total-etch and apply a suitable bonding agent according to the corresponding instructions.

Place a light cure composite according to the instructions and light cure.

Pulp vitality and status should be checked by X-ray at regular intervals.

Precautions for use

- Store MM-MTA in the sealed packaging at a dry place prior to use.
- Intraoral application of MM-MTA must be done immediately after mixing to prevent dehydration during setting.
- MM-MTA can cause discoloration. Use MM-MTA only in the root canal and/or the pulp chamber.

Storage

Store MM-MTA at a dry place at 10 – 25°C. **Do not store below 10°C!**

Do not use after expiry date.

MM-MTA Caps are for single use only.

Warranty

MICRO-MEGA® warrants this product will be free from defects in material and manufacture. MICRO-MEGA® makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and MICRO-MEGA®'s sole obligation shall be repair or replacement of the MICRO-MEGA® product.

Limitation of Liability

Except where prohibited by law, MICRO-MEGA® will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!

Caution: US Federal law restricts this device to the sale by or on the order of a dentist.

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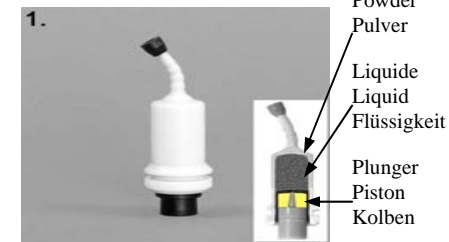
Instructions for activating and mixing MM-MTA Caps

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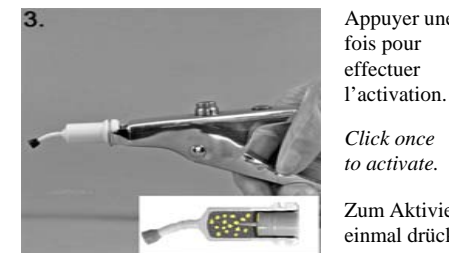
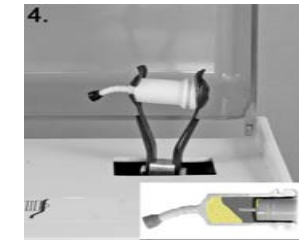
*Insert the MM-MTA Cap into the MM-MTA Gun and click **once** to activate.*

Note: The plunger must be at the same level as the bottom of the capsule.

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Insert the MM-MTA Cap into a mixer (or an amalgamator), close lid and mix immediately for 30 seconds (about 4300 oscillations / min).

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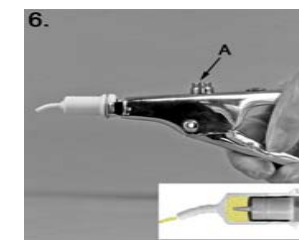
Remove the pin from the nozzle. If not, capsule can burst.

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Insert the MM-MTA Cap into the MM-MTA Gun. Pull the lever 2 times (2 clicks) to prime the MM-MTA Cap. Extrude the mixed material on a glass plate. Unlock the gun (push button A) and remove the MM-MTA Cap.

Only with the MM-MTA Gun the optimal amount of mixed material is guaranteed.

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